
# UNWIREDTHINGS

IKONN SB-1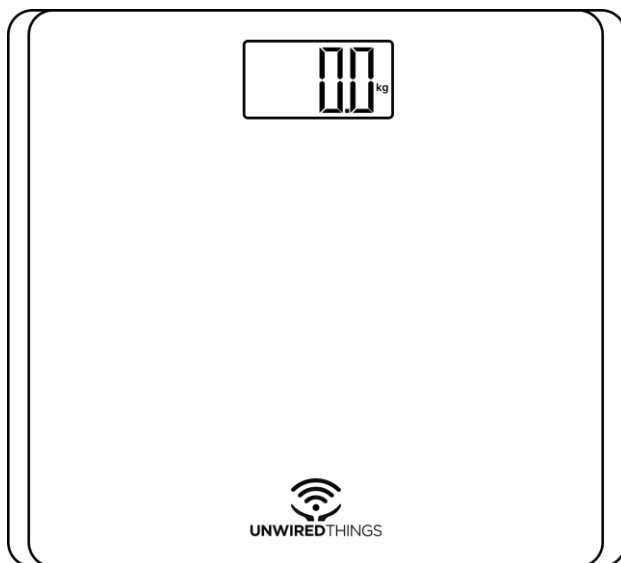

**EN** Instruction manual

**DA** Instruktionsmanual

**NO** Bruksanvisningen

**SV** Bruksanvisning

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## **EN**

---

Preface .....	3
Bathroom Scale features .....	3
Before use .....	4
Parts and components .....	4
Operation .....	5
Using bathroom scale with IKONN Controller .....	6
Tips and Troubleshooting .....	7
Handling and safety .....	8
Regulatory information .....	9

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## Preface

---

Congratulations on purchasing the IKONN SB-1 Bathroom Scale.

This bathroom scale is an IKONN-compatible product, meaning that apart from functioning as a regular scale which measures your weight - it can also be used together, with an IKONN Accessibility system which supports audio announcement of readings in your chosen language.

This manual serves as a guide to an efficient and safe use of the product. Read this information thoroughly before using the product and store this document for future reference.

**Warning:** To avoid injury or product damage, read the chapter on handling and safety, before using the product.

## Bathroom Scale features

---

The IKONN SB-1 is designed for measuring and tracking your body weight within an indoor environment.

Product features:

- System of high precision strain gauge sensors.
- Tempered glass platform.
- LCD display.
- Capacity 250 kg / 550 lb.
- Division 100 g / 0.2 lb.
- Measurement in kg/lb/st supported
- Auto On, Off and Zero functions:
  - Auto On:** The scale activates automatically when you step on it.
  - Auto Zero:** The scale automatically resets to zero before each measurement.
  - Auto Off:** After use, the scale automatically powers off to conserve battery life.
- Low battery, overload and error indications.
- Combability with the IKONN Accessibility System.

**Note:** The bathroom scale is not a diagnostic or medical device. It provides weight measurements for personal reference and should not be used to diagnose or treat medical conditions.

While connection between the bathroom scale and IKONN Accessibility System enhances daily tasks and routines for users unable to view the screen, it is not intended to replace professional medical or mobility aids. Individuals with specific medical or mobility requirements should consult their healthcare providers for guidance.

## Before use

---

Unpack the product with care.


All packaging materials are recyclable, however you may wish to keep the packaging until you are satisfied that the scale meets your needs.

Having taken the product out of the packaging, ensure that all the parts and components are present and in good condition. In case of any missing elements or if something does not seem right, please get in touch with the retailer from whom you purchased the product. Ensure that packaging bags are kept out of reach of children as these can cause suffocation.

## Parts and components

---

Item	
1	LCD Display
2	Glass platform
3	Unit switch button
4	Battery compartment
5	High precision sensor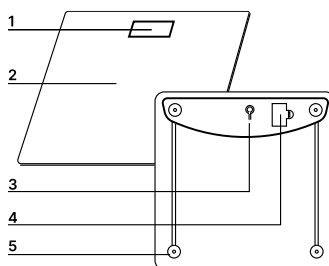

**Note:** The graphics in this manual are schematic representations used for reference only and may differ from actual product appearance.

Included components:


- IKONN SB-1
- 3 x 1.5V AAA battery
- User manual

## Operation

---

### Remove insulation strip

Open the battery compartment at the back of the scale and remove the black insulation strip. Close the battery door and wait for the LCD to display '0.0'.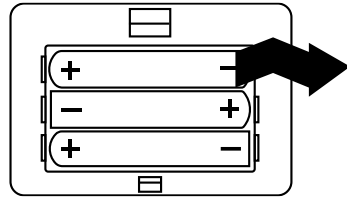

### Replace/insert batteries


To replace the batteries, open the battery compartment and remove the used batteries with the aid of a flat object. Insert new, compatible alkaline/carbon batteries – rechargeable batteries do not provide a sufficiently high voltage to support general operation. Ensure to correctly orient the batteries according to their positive and negative signs.

**Note:** Learn more about battery handling and disposal in chapter on handling and safety.

### Change measuring units

To switch between units of measure, press the 'UNIT' button placed on the back of the scale.

Pressing the button will cycle between kilograms, pounds and stone.

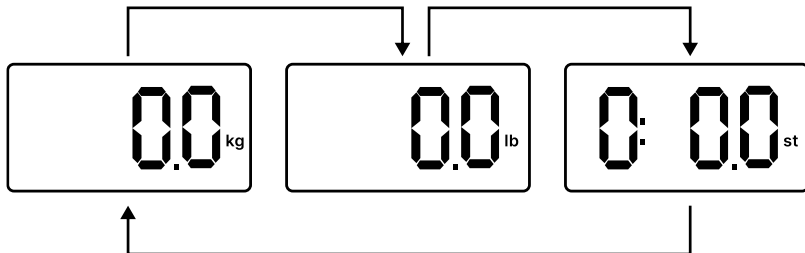## Taking measurements

Place the scale on a hard, level surface. To check that the surface is level, you can press the scale lightly with your foot and remove it – the scale should display 0.0 in your selected units of measurement.

To weigh yourself, step onto the scale. During weighing, stay still on the platform while the scale determines the weight precisely. The scale will beep when it has reached a stable measurement. Your weight will be displayed for about 40 seconds, and then the unit will switch off automatically.

**Note:** Remember to take off your shoes before taking measurements to avoid scraping or scratching the platform. The platform may be slippery when wet, therefore make sure to dry both your feet and platform before the use.

Other possible indications and their corresponding symbols are listed in the table below.

Symbol	Indication
	<b>Low battery</b> Battery power is running low. Please replace the batteries.
	<b>Scale overload</b> Weighing subject exceeds the maximum scale capacity. Please step off to avoid damage.
	<b>Measurement error</b> An error has occurred during the measurement. Please step off the scale and step onto it again to repeat the measuring process.

## Using bathroom scale with IKONN Controller

As with all Unwired Things products, this scale features the ability to connect with the **IKONN Accessibility System** which can provide audio

feedback in your chosen language. The system is intended to support users unable to view the screen for any reason. If you own an IKONN device you can also find instructions on the device for connecting to remote products.

Use the IKONN Add product function on your IKONN controller to pair your new scales with the IKONN controller. Use the help function on your controller or the manual for the controller if you are unsure how to do so.

You will hear an announcement, once the connection is established and IKONN will also state the name of the product that it has connected with. The bathroom scale function will subsequently be available in your main function list. To use the scales, with IKONN, you should now select the bathroom scale function. If the scale is off, you should turn it on immediately to allow it to connect with IKONN.

Note that more than one IKONN controller can be used to interact with the scales, though not at the same time.

You can find more support using the built-in IKONN controller help function or on the <https://ikonn.info> website.

## Tips and Troubleshooting

---

In case you are experiencing problems with the bathroom scale operation or accuracy, consider the following list of options to resolve the issue:

- Ensure that the scale is placed on a hard, level surface.
- Check each sensor foot on the back to make sure nothing is stuck to the bottom of it.
- Ensure your feet and the scale are both dry.
- Ensure that you are standing still for around 10 seconds or until the number on the display stops flashing.
- Ensure that the batteries are oriented correctly inside the battery compartment, according to their positive and negative signs and that they have sufficient charge.
- Remove the batteries to restart the cycle.

## Handling and safety

---

**Warning:** Read all safety information below before using the scale and keep this manual for future reference. Failure to follow these safety instructions could result in fire, electric shock, or other injuries, as well as damage to the product or other property.

### General safety and handling

Handle the product with care, as it contains sensitive electronic components. Always check for visible signs of damage. If you find any, reach out to the retailer from whom you purchased the product.

Operate the scale only under normal indoor temperatures. Do not operate the scale in places that are very cold or very hot, excessively dusty, where insects may enter, where it might be exposed to mechanical vibration, near flammable objects and in rooms with smoke.

Do not use a damaged scale and do not open or attempt to repair the scale yourself, as disassembling the product may cause injury to you or damage the scale and invalid warranty and proper function.

The scale should be used indoors, on flat and stable surfaces, by a single individual at a time. Always step onto the scale with bare feet or lightweight footwear. Exercise caution when stepping onto the scale to avoid slips or falls or any risk of tipping.

Ensure that the surface is clean and dry before use, clean it with a slightly damp cloth. Do not immerse the scale in water or use chemical/abrasive cleaning agents. Do not overload the scale with anything that would result in exceeding the maximum weight capacity, stated in the technical specifications. Avoid applying excessive pressure or bending the product. Never apply excessive pressure to buttons and switches. Do not drop the scale or other objects onto it.

Radio frequency emissions from the scale can negatively affect the operation of other electronic equipment.

## Batteries

Batteries must be recycled or disposed of separately from household waste. Do not dispose of batteries in normal waste and check with your authority for battery recycling advice. When handling batteries or cells, wear gloves.

Do not use deformed batteries Do not mix old with new batteries nor mix batteries of different compositions/brands. Do not burn or expose the batteries to high temperatures, as this could lead to explosions. Do not tamper with batteries or cells in any way. Avoid short-circuiting the batteries' power supply terminals and cells.

If the scale is to be stored for a long period, it is recommended to remove the batteries to avoid leakage. Avoid touching the liquid coming from leaking batteries or cells.

Ensure that batteries are out of the reach of children, especially if there's a risk of ingestion. If someone ingests a battery, seek medical help immediately.

## Regulatory information

### Technical specifications

Product	<b>IKONN SB-1</b>
Dimensions	330x300x26mm
Weight	1.34kg
Battery type	3xAAA
Capacity	250kg/550lb
Division	100g/0.2lb
Temperature	0 – 38 °C
Transmission frequency range	2402 – 2480MHz
Maximum RF output power	-1.30dBm

Technical specifications may change without prior notification to improve product quality.

Made in China.

### Disposal of electronic appliances

The product and/or batteries should be disposed of separately from household waste. Upon reaching the end of

its operational life, ensure the proper disposal of cells, batteries, or accumulators by delivering them to a collection point designated by your local authorities. Consumers are encouraged to reach out to their local authorities or the retailer for guidance on the correct disposal of old machines and their associated batteries. Adherence to these guidelines is instrumental in safeguarding the environment.

### Technical support and warranty

Unwired Things assumes responsibility to the end user or consumer for any lack of conformity that may exist at the time of product delivery, in accordance with the terms, conditions, and deadlines stipulated by the relevant

regulations. All Unwired Things products are covered by a two year warranty.

We recommend that all repairs be conducted by qualified personnel.

Should you encounter any issues with your product or have any uncertainties at any point, please do not hesitate to reach out to the retailer from whom you purchased the product.

### Copyright

Unwired Things ApS retains the intellectual property rights for the content contained within this document. All rights are reserved. Unauthorized reproduction, distribution, or use of any part of this document and product may violate applicable laws and could result in civil and criminal penalties.

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